

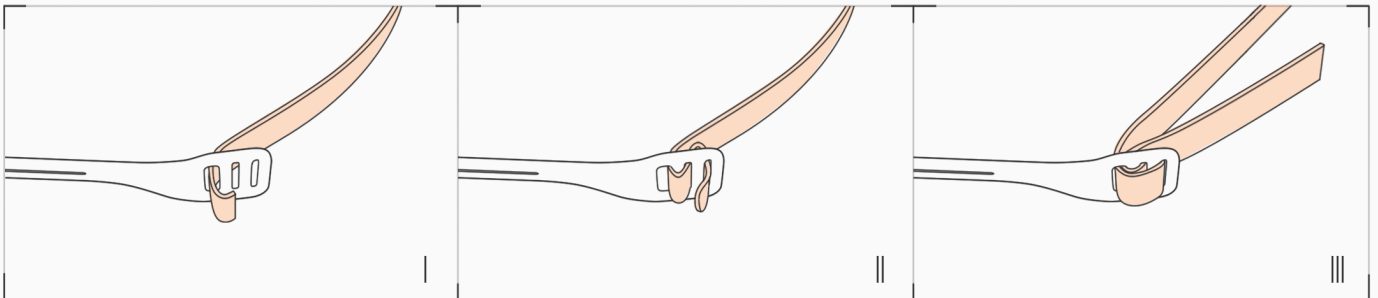


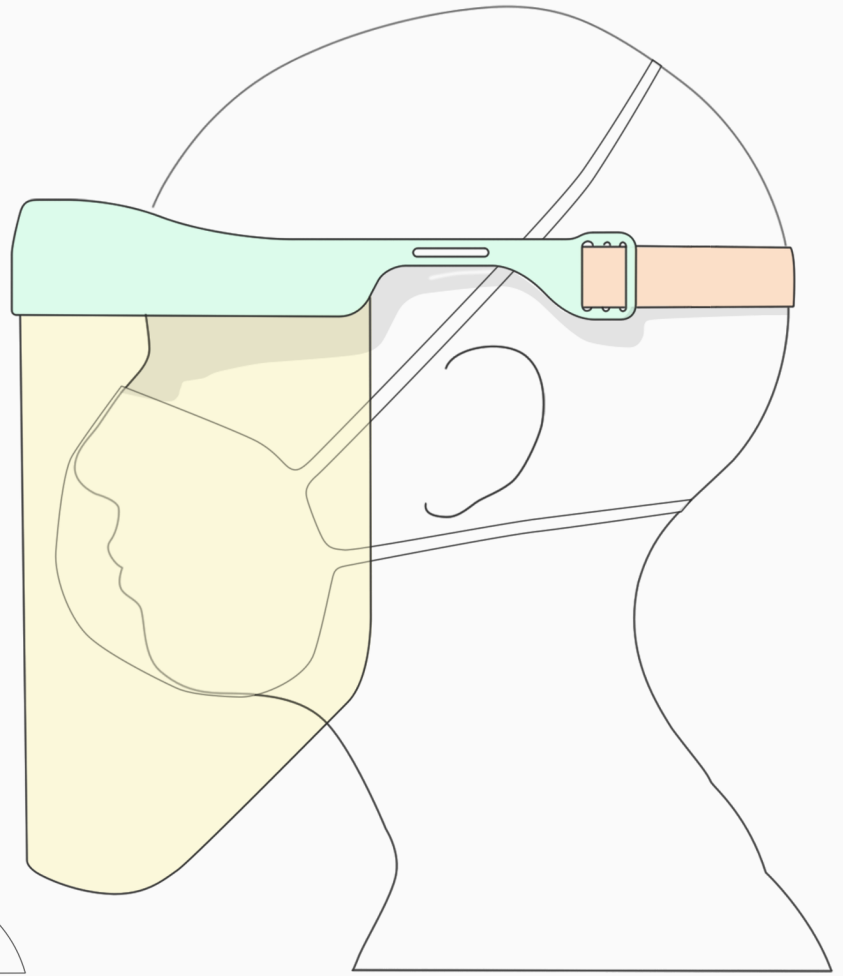

RE-USABLE FACE SHIELD



3D-printed protective equipment for medical personnel to assist the NHS during the international COVID-19 pandemic.

Please follow cleaning guidelines as defined by your local working practices but it is suggested that the item is cleaned regularly with antibacterial wipes.

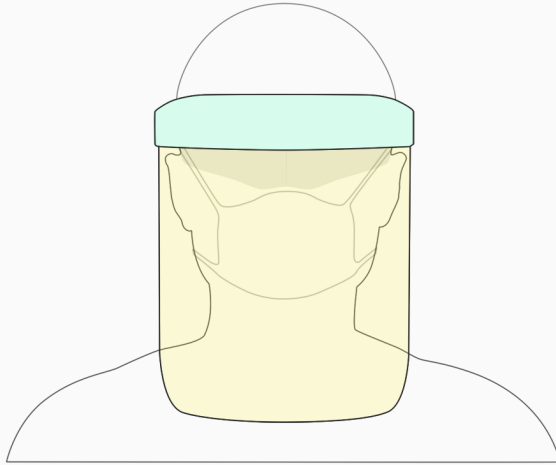

Thread the elastic ribbon through the vertical slots on one side of the headpiece, as shown in the three stage illustration. Repeat on the other side of the headpiece. The elastic ribbon should be secure on both sides.

Adjust elastic ribbon by pulling through the loops to obtain a secure yet comfortable fit.

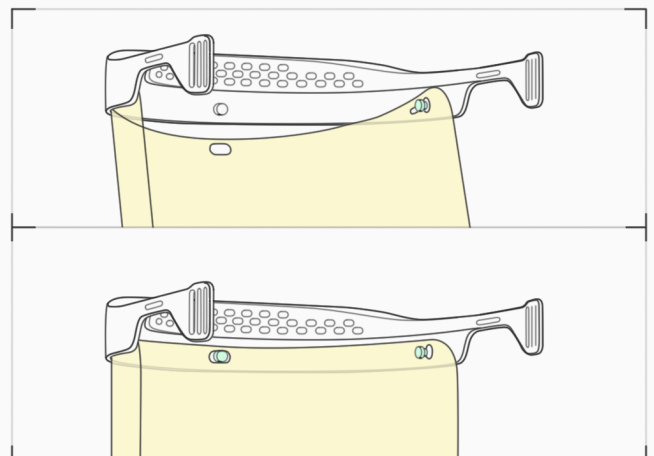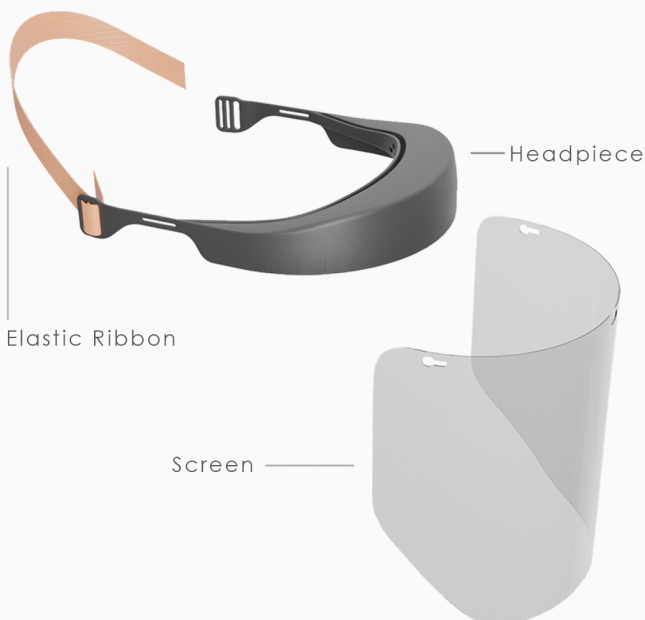

Holding the headpiece so the pins are easily accessible, gently bend the pre-cut clear plastic screen before letting go of the screen so the pins slide into their engaged position. Slowly rotate and position the screen, so that the centre peg on the headpiece protrudes through the centre slot on the screen.